

SleepStyle™ 200

CPAP SERIES

Ambient
Tracking®
Plus

Specification Sheet

A Versatile CPAP Solution



The SleepStyle™ 200 CPAP Series with innovative design delivers:

- *Unique auto-adjusting humidification technology*
- *Improved CPAP performance*
- *Better clinical, patient-care and commercial results*

Performance + Versatility = Results

The **SleepStyle™ 200 CPAP Series** – the all-new versatile family of standard and integrated CPAPs deliver sleep performance through improved **clinical, patient-care** and **commercial results**.

Ambient
Tracking®
Plus

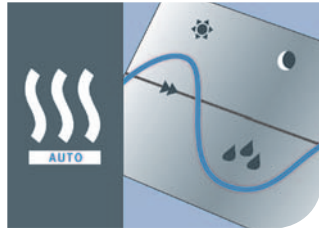



Figure 1: Mask pressure with and without condensate in tubing (adapted from Bacon et al, 2000)

CLINICAL RESULTS

A common side effect of conventional humidifiers is the accumulation of water (condensate) in the breathing tube, resulting in disruptive gurgling noises, tube vibrations and mask pressure instability. Research has shown as little as 10ml of condensate causes a 5.6cm H₂O inspiratory pressure drop at the mask.¹

Ambient Tracking® Plus balances humidity to provide the most effective CPAP therapy under changing conditions by auto-adjusting humidity to compensate for:

- Changing temperatures – minimizes condensation
- Humidity loss during leaks (occur up to 30% of sleep time²) – maximizes humidity

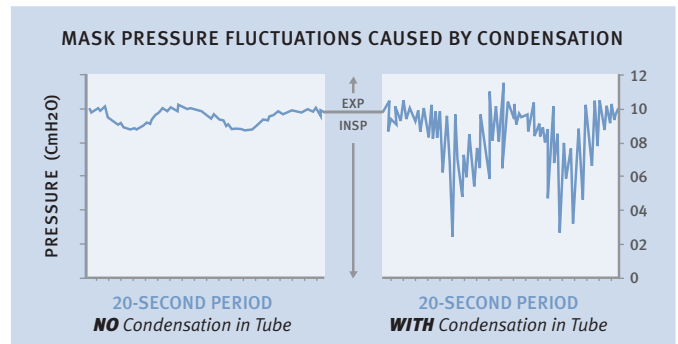

PATIENT-CARE RESULTS

The **SleepStyle™ 200 CPAP Series** has been designed with **integrated simplicity** in mind. The ‘truly integrated’ compact design is a common feature of all standard and humidified models within the **SleepStyle™ 200 CPAP Series**.

This simplicity ensures quick and easy:

- Set-up and use
- Conversion to cold or heated humidification
- Cleaning and maintenance

An intuitive navigation menu is common across all models. Providers and patients can upgrade therapy conveniently using an electronic key. Travel needs are satisfied with a **compact and robust** CPAP that adjusts altitude and voltage anywhere in the world.



HC013 Cold Conversion Kit



HC014 Heated Conversion Kit

COMMERCIAL RESULTS


The SleepStyle™ 200 CPAP Series is designed to save time and money for your business.

Time-saving features that minimize set-up times and patient follow-up issues include:

- Versatile therapy allowing conversion from standard CPAP to cold or heated humidification (electronically) at any time during treatment
- Simple one-piece chamber design
- Intuitive navigation menu including one-step operation to adjust humidity
- Patient compliance monitoring generates a one-page report via a quick telephone call, or a more comprehensive report by machine download

Value-added features associated with improving compliance:

- Ambient Tracking® Plus – patented auto-adjusting humidification technology
- Proportional Ramp – short term pressure relief to allow initiation of sleep

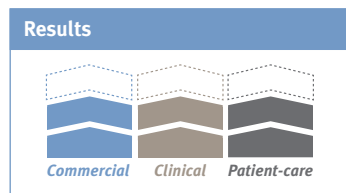

Our business, like yours, is all about results. Optimal results come from a range of high-performance features.



The table below shows the range of performance features available in the SleepStyle™ 200 CPAP Series

SleepStyle™ 200 Series			Units*					
Performance Category	Performance Feature		231	232	233	234	236	238
Humidification Delivery	A	Convertible/Reversible System (Cold Passover/Heated Humidification)	●	●			●	
	B	Fully Integrated System			●	●		●
	C	Ambient Tracking® Plus			●	●		●
Performance Logging	E	Compliance Maximizer™ Software		●		●	●	●
	F	Titration Maximizer™ Software	●	●	●	●	●	●
Pressure Control	G	Altitude Adjustment Control - Manual	●	●	●	●		
		Altitude Adjustment Control - Automatic					●	●
	H	Leak Compensation					●	●

*Some models not available in all countries

KEY	◀ . . . ▶	Converts to this model using conversion kit	HC013	Conversion Kit – Cold Humidification
			HC014	Conversion Kit – Heated Humidification

SleepStyle™ 200

CPAP SERIES



- **PERFORMANCE** - CPAP performance has been improved through increased pressure range and reduced noise levels.
- **AMBIENT TRACKING® PLUS** is a unique technology that promotes better humidification and clinical performance by superseding conventional humidifiers.
- **VERSATILITY** - Therapy can convert from standard CPAP to cold or heated humidification at any time during treatment.
- **EASE OF USE** - All new intuitive menu, quiet operation and compact design add up to a quick and easy provider set-up.

The SleepStyle™ 200 CPAP Series will help you provide 24-hour wellness for your patients. Advanced technologies deliver the best in sleep performance for an energized lifestyle. For more information or to view our complete SleepStyle™ CPAP family, visit www.fphcare.com.



SLEEPSTYLE™ 200 CPAP SERIES PACK INCLUDES:

- CPAP Unit
- Carry Bag
- 2 x Water Chamber
- Breathing Tube
- Air Filter
- Water Funnel

ACCESSORIES

- | | |
|----------|--|
| HC013 | Conversion Kit – Cold Humidification |
| HC014 | Conversion Kit – Heated Humidification |
| HC355* | Extended Life Humidification Chamber |
| HC385S | Standard Humidification Chamber |
| 900HC240 | Air Filter (2 pack) |
| 900HC221 | Tubing including connector |
| 900HC230 | Compliance Maximizer Software |
| 900HC232 | Compliance Download Serial Cable |
| 900HC607 | Titration Maximizer Software |
| 900HC236 | Isolated Serial Adaptor |
| 900HC226 | Lead Extension Kit |

PRODUCT SPECIFICATIONS

- | | |
|----------------------------|--|
| Dimensions: | 10.8" x 6.7" x 5.5"
(275mm x 170mm x 140mm) |
| Weight: | 4.4 lbs (2kg) |
| Chamber Capacity: | 400ml |
| Electrical Ratings: | Supply Frequency: 50-60Hz
Voltage and Current: 100-115V ~ (1.2A, 1.3A max)
220-240V ~ (0.8A, 1.8A max) |
| Operating Pressure: | 4-20cm H2O |



SleepStyle™
CPAP SERIES STANDARD



SleepStyle™
CPAP SERIES INTEGRATED

* Not available in all countries

REFERENCES: 1. Bacon JP, Farney RJ, Jensen RL et al: Nasal continuous positive airway pressure devices do not maintain the set pressure dynamically when tested under simulated clinical conditions. Chest 2000; 118 (5):1441-9. 2. Martins De Araujo MT, Vieira SB, Vasquez EC et al: Heated humidification or face mask to prevent upper airway dryness during continuous positive airway pressure therapy. Chest 2000; 117 (1):142-7.

FOR MORE INFORMATION, PLEASE CONTACT YOUR LOCAL FISHER & PAYKEL HEALTHCARE REPRESENTATIVE:

Fisher & Paykel Healthcare has a policy of continuous product improvement and reserve the right to alter specifications without notice. Patented in principal countries around the world.

International P O Box 14 348, Panmure, Auckland 1134, New Zealand Tel: +64 9 574 0100 Fax: +64 9 574 0158 Email: info@fphcare.com Web: www.fphcare.com **USA** Tel: 1800 446 3908 +1 949 470 3900 Fax: +1 949 470 3933 **Australia** Tel: +61 3 9879 5022 Fax: +61 3 9879 5232 **Austria** Tel: 0800 29 31 23 Fax: 0800 29 31 22 **Benelux** Tel: +31 78 644 0924 Fax: +31 78 644 0914 **China** Tel: +86 20 8363 4286 Fax: +86 20 8363 3595 **France** Tel: +33 1 6446 5201 Fax: +33 1 6446 5221 **Germany** Tel: +49 7182 93777 0 Fax: +49 7182 93777 99 **India** Tel: +91 80 5123 6041 Fax: +91 80 5123 6044 **Irish Republic** Tel: 1800 409 011 **Italy** Tel: +39 06 7839 2939 Fax: +39 06 7814 7709 **Japan** Tel: +81 3 3661 7205 Fax: +81 3 3661 7206 **Spain** Tel: +34 902 013 346 Fax: +34 902 013 379 **Switzerland** Tel: 0800 83 47 63 Fax: 0800 83 47 54 **UK (EU Authorised Representative)** Tel: +44 1628 626 136 Fax: +44 1628 626 146 **Nthn Ireland** Tel: 0800 132 189